


Please refer to the **Hologic ThinPrep LBC Pap Sample Collection Instructions**
**Please note expiry date on vial*

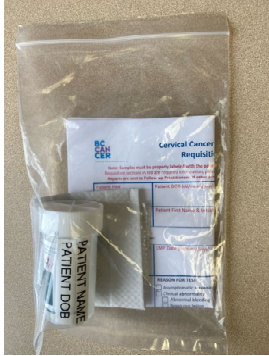

Ensure cap on specimen vial is tightly closed
by lining up black lines on vial and lid

Place sample vial in sealable plastic bag (one sample per bag only) + absorbant pad

- Absorbant pad examples:
- paper towel
 - gauze square
 - cotton pad

Fold completed associated CCSL requisition and place in bag


Keep sample vial at 15-30°C

Shipping qualifies for "Excepted Quantities" as long as each inner bag contains 1 vial (20ml) and no more than 25 vials per outer package (max 500ml)

Place up to 25 individual bags in outer packaging and mark for delivery to CCSL

- Outer package examples:
- envelope, bag
 - cardboard box
 - other as per courier pickup
- LABEL OUTER PACKAGE "CCSL" with address (below)**

Select transport method as established

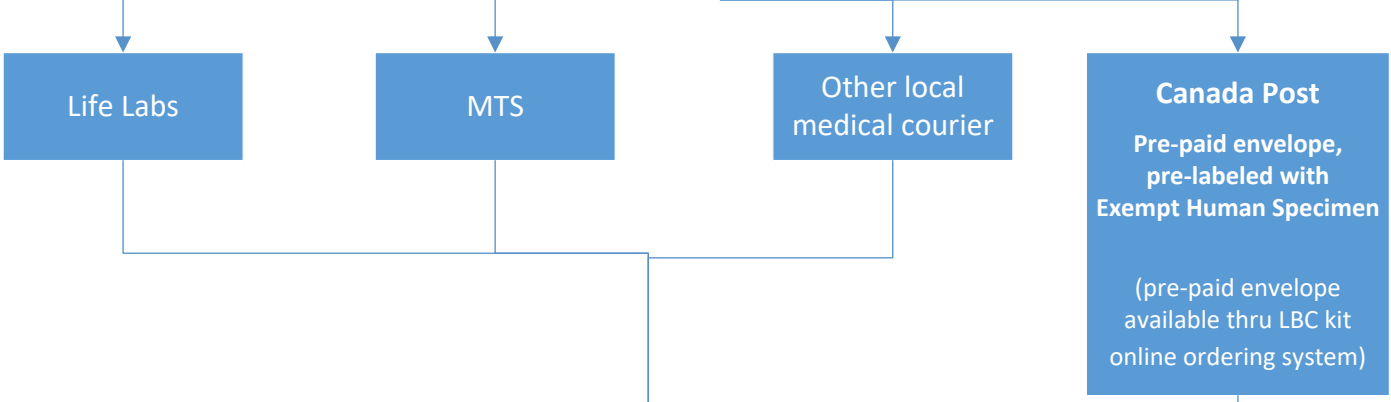

Deliver to
CCSL, Central Processing & Receiving
655 W12th Avenue
Vancouver, BC
V5Z 4R4